



HP Confidential. For use by HP or Partner with Customers under HP CDA only.

Please refer to the HP Healthcare Edition HC271/HC271p Clinical Review display user guide for instructions on cleaning this product. Here are additional instructions for cleaning with specific germicidal wipes.

- Turn off the display.
- Disconnect any external devices.
- Dust the display by wiping the screen and the cabinet with a soft, clean antistatic cloth.

IMPORTANT: The cloth should be moist, but not wet. Water dripping into the ventilation or other points of entry can cause damage to the display. Allow the unit to air-dry before use.

Next, use any of the following germicidal wipes* to clean all exposed surfaces of your display. Refer to the directions for use provided by the manufacturer of the wipes.

- **Sani-Cloth® HB**
 - Benzyl-C12-18-alkyldimethyl ammonium chlorides: <0.1%
 - Quaternary ammonium compounds, C12-14-alkyl[(ethylphenyl)methyl]di methyl, chlorides: <0.1%
- **Sani-Cloth® Plus**
 - Isopropanol: 10-20%
 - 2-Butoxyethanol: <5%
 - Quaternary ammonium compounds, C12-18-alkyl[(ethylphenyl) methyl] dimethyl, chlorides: <0.5%
 - Benzyl-C12-18-alkyldimethyl ammonium chlorides: <0.5%
- **Sani-Cloth® AF3**
 - Quaternary ammonium compounds, C12-18-alkyl[(ethylphenyl) methyl] dimethyl, chlorides: <0.5%
 - Benzyl-C12-18-alkyldimethyl ammonium chlorides: <0.5%
- **Super Sani-Cloth® Germicidal Disposable Wipes**
 - Isopropyl alcohol: 55.0%
 - Alkyl dimethyl benzyl ammonium chlorides: 0.250%
 - Alkyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium chloride: 0.250%
- **CaviWipes®**
 - Isopropanol: 10-20%
 - Ethylene Glycol Monobutyl Ether (2-Butoxyethanol): 1-5%
 - Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride: 0.1-0.5%
- **Clorox Healthcare® Bleach Germicidal Wipes**
 - Sodium hypochlorite 0.1%–1.0%
- **Oxivir® Tb Wipes**
 - Hydrogen Peroxide: 0.1-1.5%
- **Windex® Electronic Wipes**
 - Cellulose: 10-30%
 - Ethyl alcohol: 0.10-1.00%
- **PDI Easy Screen® Cleaning Wipe**
 - Isopropyl alcohol: 30-40%
 - Water: 60-70%

Please do not use cleaners that contain any petroleum-based materials such as benzene, thinner, or any volatile substance to clean the display screen or cabinet. These chemicals may damage the display.

Sincerely,

© COPYRIGHT 2018 * DISCLAIMER: Solutions tested as of May 15, 2018; chemical composition is subject to change. Repeated use of these germicidal wipes may cause some cosmetic changes to the product. 4AA7-3876ENW